CEINTURE ABDOMINALE COOL EFFECT

CINTURÓN ABDOMINAL EFECTO FRÍO - COOL EFFECT ABDOMINAL BELTS - COOL EFFECT BUIKGORDEL

REF: 580021 N° DE LOT: 12-21/01

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EN : This appliance can be used by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision. Children should not play with the appliance.

- Do not use on sensitive parts of the body (mucous membranes, eyes, etc.).
- Do not heat in the microwave.
- Do not place in the freezer.
- Keep sharp objects away from ice packs and avoid placing heavy objects on them, as this may cause them to burst.
- Do not place on a hot surface. Do not open, do not pierce, do not tear, do not put in the mouth, in case of spillage on the skin, rinse thoroughly, and in case of contact with eyes, consult a doctor.
- If the liquid is swallowed, consult a doctor or call the poison control centre immediately and make sure you keep the leaflet so that you have the composition of the gel.

⚠ NOT FOR THERAPEUTIC/MEDICAL USE.

RESTRICTIONS ON USE : Do not use the gel pad if: you have circulation problems, have limited temperature sensitivity, are taking prescription painkillers, have open wounds or damaged skin. Do not touch the eyeballs during use and take care not to get the gel in the eyeballs, use with caution for patients with sensitive skin.

FOR PATIENTS OVER 18 YEARS OLD AND UNDER 55 YEARS OLD.

Caution : These compresses are for external use only. They should be kept out of the reach of children. They should be used by adults between 18 and 55 years of age. Before use, check that the pad is not damaged. If it is, please throw it away. Keep away from open flames and heat sources.

COLD COMPRESS : Never use this compress without its protective cover, as this would expose the patient to the risk of frostbite.

In case of doubt and in case of particular health problems, it is recommended to seek the advice of a health professional.

Cryotherapy should be prescribed for people with a number of conditions and diseases.

Among those presenting contra-indications, it is important to quote:

People who have had a stroke - People with uncontrolled high blood pressure - Respiratory insufficiency - Circulatory insufficiency - Angina pectoris - People with a pacemaker - Stage 3 or 4 arterial disease - Deep vein thrombosis Deep vein thrombosis - Acute respiratory infections - Nephritic colic - Deep anaemia - Allergies to cold and Raynaud's syndrome - Acute bacterial or viral cryoglobulinaemia - Acute deep infections - Anticoagulant use - In certain cases, in some cases, the contraindications are relative and require a thorough interrogation or the agreement of the doctor. This is particularly the case for the following diseases: Cardiac rhythm disorders - Valvular insufficiency - Valvular narrowing - Stage 1 and 2 arteriopathy - Aeschaemic heart disease - Polyneuropathy - Cases of claustrophobia.

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EN : REMINDER :

- **Only for cold use. Do not heat.**
- **Do not freeze.**

1. Place the ice packs flat in the refrigerator (in a temperature above 0°C) for at least 2 hours. **1**
2. Then put them in the nets inside the belt. **2**

NOTE: Check that the pockets are not too cold to avoid injury from cold.

3. Fasten it to your thigh with the hook and loop fastener. **3**
- Make sure you put the side with the nets against your stomach.

NOTE: Suitable for waist sizes up to 85cm.

4. The maximum duration of use is 20 minutes to avoid injuries due to the cold.
5. Put the gel pack back in the fridge for future use.

PRECAUTIONS :

- **Do not use for more than 20 minutes and wait at least 20 minutes before reapplying.**
- **If the ice packs are frozen, leave them at room temperature for a few minutes, until they are less frozen.**
- **It is recommended to use the cold belt in a room temperature environment to avoid cold reactions.**

BEAUTY ROUTINE : do a session of about 15 minutes, every other day.

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EN : Wash the product before first use.

- **Cleaning the cover :** Wash it by hand in cold water (less than 30°C). The washing time should not be too long (3 ~ 5 minutes) to avoid shrinkage, damage to the belt and colouring.

- **Cleaning the liquid pockets :** wash them gently with soap and rinse with cold water. Wipe with a soft cloth. Store the product in a dry place, away from direct heat and humidity.

Caution! Do not puncture liquid pockets.

- **Disposal instructions :** When the time comes, please dispose of the product in an environmentally friendly manner via an authorised disposal centre or your local waste centre. Observe the applicable regulations on the product and packaging.

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